

Accessory Basket with Capsule Neuron 2

Quick Installation and Maintenance Instructions



CS-SLVP-ADA-00

Medical Facility Responsibility

WARNING: It is the responsibility of the end user to ensure all aspects of installation are covered by following the full manual found at <https://www.amico.com/products/equipment-mounts/patient-monitor/capsule-monitor-mounts/capsule-neuron-mounts>.

Installation Tools



1x #2 Phillips Head Screwdriver



1x #3 Phillips Head Screwdriver



Hex Key: 1/8"

Installation Reference



NOTE: Tools for installation not provided by Amico Accessories. Verify the weight of the monitor to be mounted. This product is rated at 20 lbs total, including 5 lbs in the basket. The *CAPSULE NEURON 2* must be mounted with the *NIBP PACK** as shown in **Figure 2**. The *CAPSULE NEURON 2* can only be mounted to *CS-SLVP-ADA-00* without the *NIBP PACK* in a non-flush configuration.

Mounting to AHM- 4-_/SSM- 4-_

Insert 3x *FLAT HEAD SCREWS* through the top of the *MOUNTING PLATE* and into the *DISK* as shown in **Figure 1** using a 1/8" Hex Key/bit. Use the provided 3/16" Hex Key to tighten and secure.

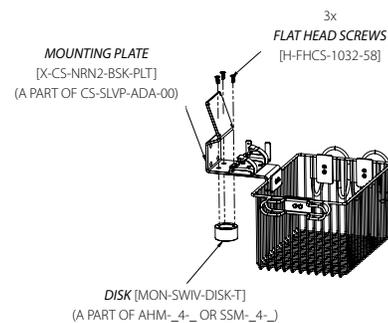


Figure 1

For Use with Capsule Neuron 2 (Vitals Plus)

Insert 4x *PAN HEAD SCREWS* through 4x *LOCK WASHERS* through the back of the *MOUNTING PLATE* and into the device (*CAPSULE NEURON 2* and *NIBP PACK**) as shown in **Figure 2**.

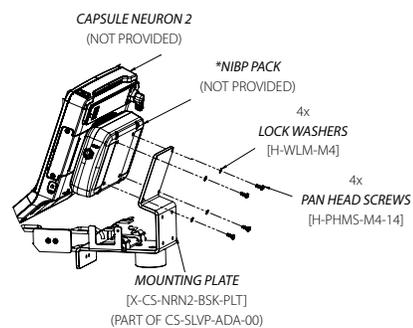


Figure 2

Accessory Basket with Capsule Neuron 2 Quick Installation and Maintenance Instructions



CS-SLVP-ADA-00

For Use with Power Supply Mount

1. Release the **STRAP** by opening the **BUCKLE** and pulling the **STRAP** out and ensuring that the **LOOP** is still on the **STRAP** as shown in **Figure 3**.
2. Place the **POWER SUPPLY** within the **STRAP** as shown in **Figure 3**.
3. Hold the **POWER SUPPLY** in position and insert the end of the **STRAP** into the **BUCKLE** and pull out to tighten the **POWER SUPPLY** before closing the **BUCKLE** as shown in **Figure 4**.
4. Dress the end of the **STRAP** through the **LOOP** and the **CABLE** into the **CABLE CLIP** as shown in **Figure 4**.
5. Repeat steps 3 and 4 for the second **STRAP**.

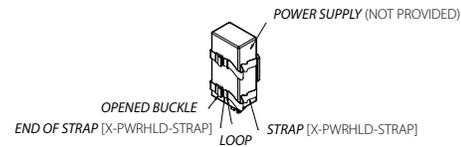


Figure 3



Figure 4

Troubleshooting Guide

Symptom	Possible Cause	Solution
Capsule Neuron 2 Vitals Plus is loose	Improper or loose mounting configuration to Amico mount	Tighten set screws and refer to instructions (For Use with Capsule Neuron 2 Vitals Plus)
The basket is loose	Improper or loose mounting configuration to Amico mount	Refer to instructions (Mounting to AHM/SSM)

Maintenance

It is recommended that once every three (3) months, all locks, bolts, and screws are visually inspected for signs of product wear and tear or damage. Check the product for looseness in the mount or orientation, and ensure the adapter is secured to the rail or wall channel system. Ensure the set screws on the channel and (2) thumb screws are fastened to the tightest possible position and ensure the device is secure.

Warranty

During the term of warranty: Within the first twelve (12) months from the date of shipment, Amico Accessories will repair or replace any part which is proven to be defective at no cost. After the twelve (12) month period, Amico Accessories will send the parts to the customer free of charge, however, shipping and installation will be borne by the customer.

The warranty is valid only when the product has been properly installed according to Amico Accessories specifications, used in a normal manner, and serviced according to factory recommendations. It does not cover failures due to damage which occurs in shipments or failures which resulted from accidents, misuse, abuse, neglect, mishandling, alteration, misapplication or damage that may be attributable to Force Majeure.

AMICO ACCESSORIES DOES NOT HONOR VERBAL STATEMENTS CONCERNING THE WARRANTY.

The distributor and/or dealer are not sanctioned to create verbal warranties about the product described in this agreement. Any statements will not be honored or be made part of the agreement of sale. This document is the final complete and exclusive terms of the agreement.

THIS WARRANTY IS INCLUSIVE AND REPLACES ALL OTHER WARRANTIES.

Amico Accessories shall not, under any circumstances be liable for incidental or consequential damages including, but not limited to, profit, loss of sales or injuries to person(s) or property.

Correction of non-compliance as noted above will result in completion of all liabilities of Amico Accessories whether based on agreement, neglect or changed materials, designs or specifications without notice.

All claims for warranty must first be approved by Amico Accessories Customer Service Department: (info@amico-accessories.com or 1-877-264-2697). A valid Return Goods Authorization number must be obtained from Amico Accessories prior to commencement of any warranty claim.