

Exergen Thermometer Mount

Quick Installation and Maintenance Instructions



Medical Facility Responsibility

Preventive maintenance checks must be performed regularly to maintain the quality and performance of this product. Any parts that may be broken, missing, worn, distorted, or contaminated in any way should not be used and all affected parts should be replaced immediately. Should the necessity of any repair be suspected; please contact your local distributor.



WARNING: It is the responsibility of the end user to ensure all aspects of installation are covered by following the full manual and reviewing warnings/notes.

Exergen TAT-5000 Wall Mount



Exergen 134201



Exergen 134305/134307
Exergen 134306 (not shown)
Exergen 134308 (not shown)

Installation Tools



#2 Phillips Head Screwdriver



#3 Phillips Head Screwdriver

Installation to Mounting Platform Instructions


1. Place the *X-XG-INHLD-BKT* with relevant mounting adapter *ATA/FTA/VTA/PC-125* to the respective *ARS/FRS/VRS/RS RAIL*.
2. Refer to **Figure 1** for installing rail adapter to *EXERGEN 134305/134306/134307/134308*, and **Figure 2** for installing *RAIL ADAPTER* to *EXERGEN 134201*.
3. Refer to the "Section 4: Installation on Mounting Platform" of the Barcode Dock Mount manual for instructions on mounting connecting an adapter to a rail:
 - ARS Rail – "Mounting on ARS" under section 4
 - FRS Rail – "Mounting on FRS and LRS" under section 4
 - VRS Rail – "Mounting on the VRS/Dovetail/Ohmeda Rail" under section 4
 - Roll stand 1.25" post - "Mounting on 1.25" Diameter Post" under section 4

If you haven't received an instruction for your rail system, please contact accessories@amico.com.


Exergen Thermometer Mount

Quick Installation and Maintenance Instructions

Exergen 134305/307

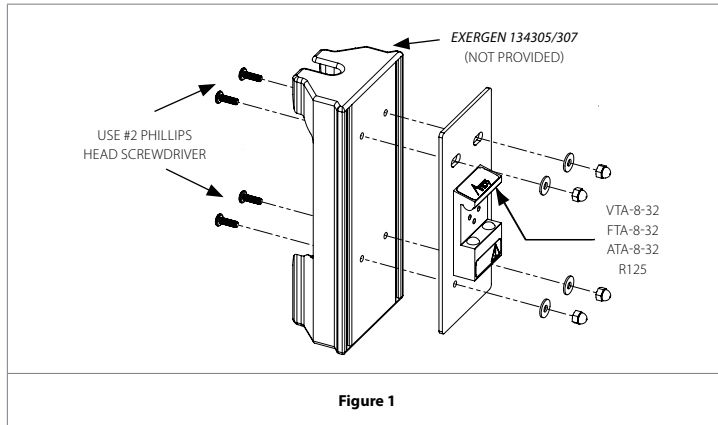

Exergen 134201

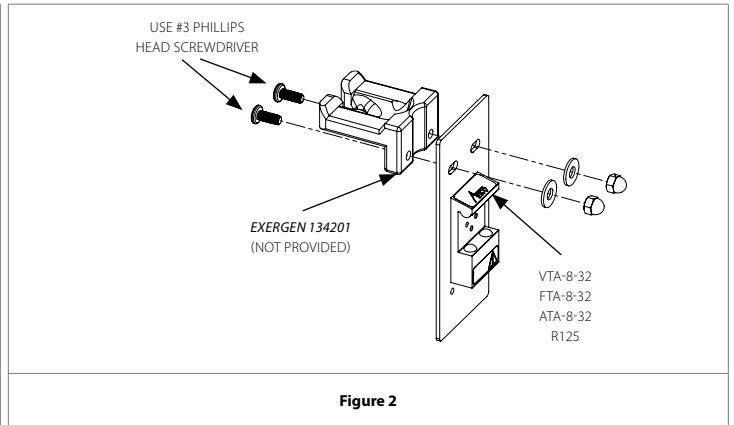

Maintenance

It is recommended that once every three (3) months, all locks, bolts, and screws are visually inspected for signs of product wear and tear or damage. Check the product for looseness in the mount or orientation, and ensure the adapter is secure.

Warranty

Amico Accessories Inc. warrants all mounting accessories to be free from defects in material and workmanship for a period of twelve (12) months from the date of shipment. Within this period Amico Accessories Inc. will repair or replace any part which is proven to be defective.

Amico Accessories Inc. will warrant its materials to be free from defect for an additional period of four (4) years, (five [5] years from the date of shipment). Within this period, Amico Accessories Inc. will replace any part which is proven to be defective, at no charge. Shipping and Installation costs after the first twelve (12) months will be borne by the Customer.

This warranty is valid only when the product has been properly installed according to Amico Accessories Inc. specifications, used in a normal manner and serviced according to factory recommendations. It does not cover products that are not manufactured by Amico Accessories Inc. It does not cover failures due to damage which occurs in shipments or failures which result from accidents, misuse, abuse, neglect, mishandling, alteration, misapplication or damage that may be attributable to acts of God.

Amico Accessories Inc. shall not be liable for incidental or consequential damages resulting from use of equipment.

All claims for warranty must first be approved by Amico Accessories Inc. A valid Return Goods Authorization (RGA) number must be obtained from Amico Accessories Inc. prior to commencement of any service work. Warranty work, which has not been pre-authorized by Amico Accessories Inc., will not be reimbursed.

AMICO ACCESSORIES INC. DOES NOT HONOR VERBAL STATEMENTS CONCERNING THE WARRANTY.

The distributor and/or dealer are not sanctioned to create verbal warranties about the product described in this agreement. Any statements will not be honored or be made part of the agreement of sale. This document is the final complete and exclusive terms of the agreement.

THIS WARRANTY IS INCLUSIVE AND REPLACES ALL OTHER WARRANTIES.

Amico Accessories Inc. shall not, under any circumstances be liable for incidental or consequential damages including, but not limited to, profit, loss of sales or injuries to person(s) or property. Correction of noncompliance as noted above will result in completion of all liabilities of Amico Accessories Inc., whether based on agreement, neglect or changed materials, designs or specifications without notice.

All claims for warranty must first be approved by Amico Accessories Inc. Service Department: accessories@amico.com or 1.877.264.2697. A valid Return Goods Authorization number must be obtained from Amico Accessories Inc. prior to commencement of any warranty claim.