

# Powered Cart with Standard Interface

## Quick Setup, Installation, and Adjustment Instructions



### Automatic Height Adjustment

#### Medical Facility Responsibility

**WARNING:** This Quick Guide is not a replacement for the full manual. It is the responsibility of the end user to ensure all aspects of the installation are covered by following the full manual.

**NOTE:** Inspect the cart integrity prior to beginning setup.

#### 1. First Time Initializing:

- Briefly plug the cart into the power outlet. The AC Power Indicator should illuminate (the may take up to 30 seconds).
- After initializing the cart, the two arrow buttons (**Figure 1**) will begin to flash, indicating that the user must set the cart's "Home" (lowest) position.

#### 2. Setting Home Position:

- Hold the Down arrow to lower the cart's worksurface until only the Down arrow is flashing.
- Release the Down arrow and reverse direction to raise worksurface to desired position.

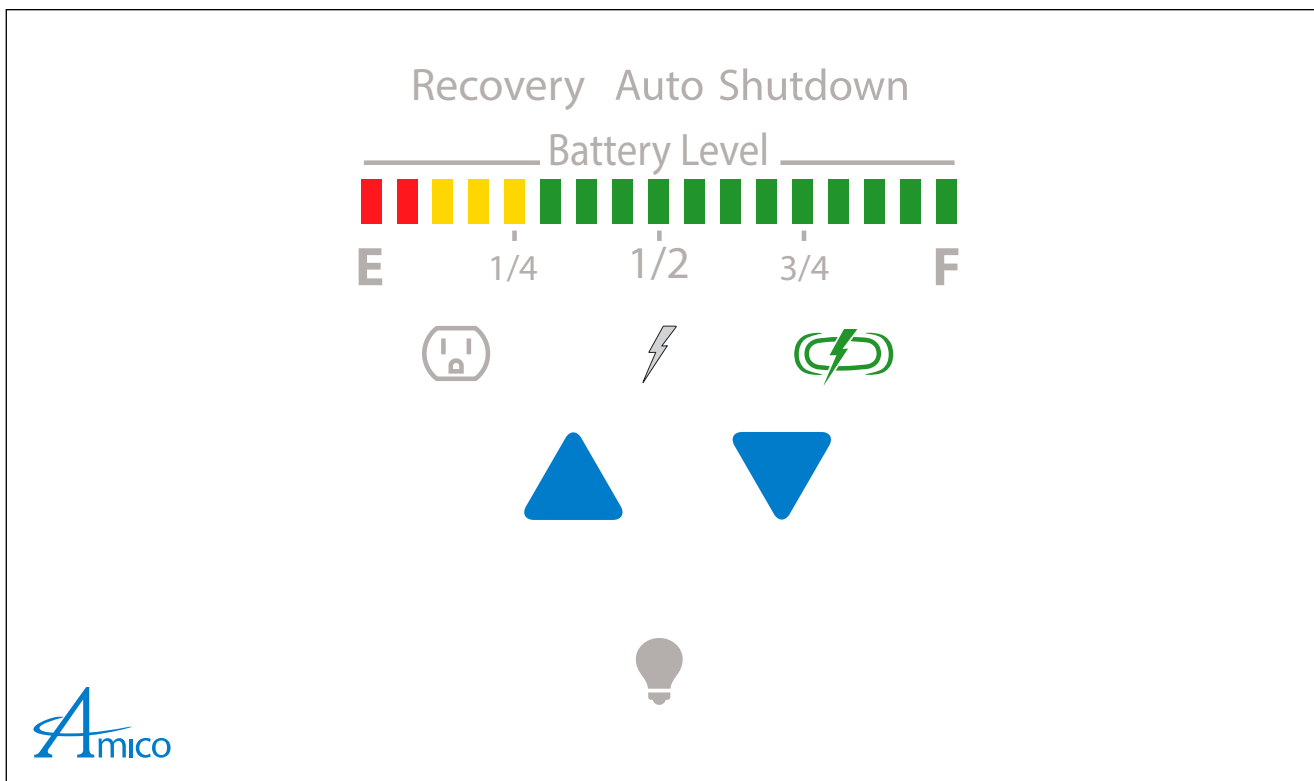


Figure 1

# Powered Cart with Standard Interface

## Quick Setup, Installation, and Adjustment Instructions



### Manual Height Adjustment


#### Medical Facility Responsibility

**WARNING:** This Quick Guide is not a replacement for the full manual. It is the responsibility of the end user to ensure all aspects of the installation are covered by following the full manual.

**NOTE:** Inspect cart integrity prior to beginning setup.

#### 1. First Time Initializing:

- Briefly plug the cart into the power outlet. The AC Power Indicator should illuminate (this may take up to 30 seconds).

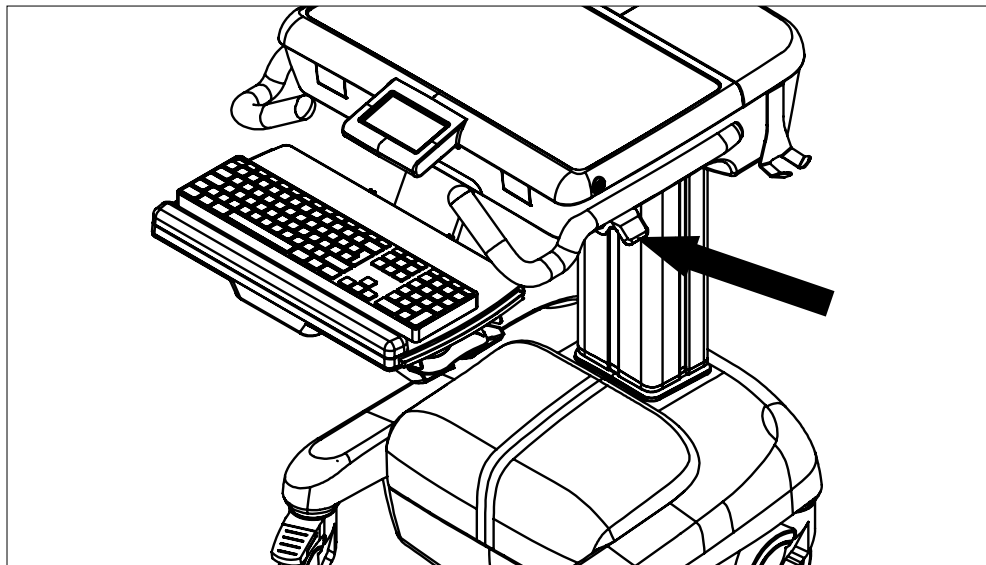

Manual Height Adjustment

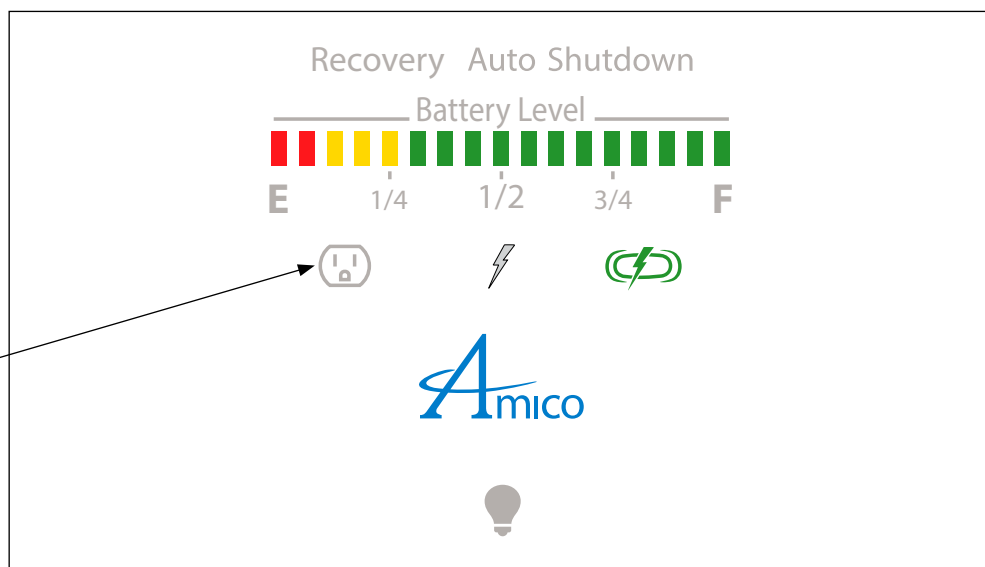


Figure 1