


Installation Instructions

Service Chase

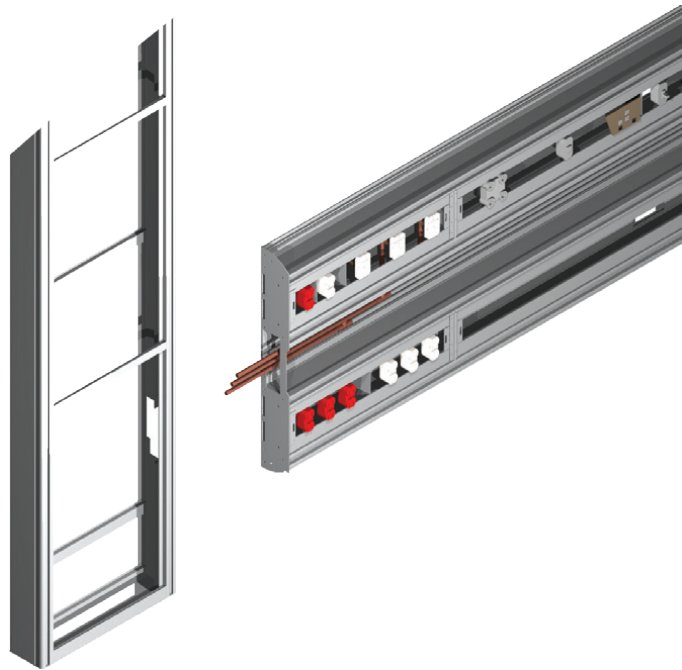

Note

Service chase installation should conform to local building codes

Before Installation


1. Review a copy of the final approved shop drawing(s) and/or submittal package. These documents will provide you with technical information specific to your installation, such as:
 - Equipment types and quantities
 - Room numbers and location
 - Shop drawings of each unit type
 - Wiring diagrams and rough-in locations
2. Locate the chase from the shipping boxes that you wish to install.
The contents of each shipping box are listed on the shipping document(s).

Installation


1. Remove laminate panels from chase (using tool provided) by lifting panels vertically until resistance is felt, then rotate bottom of panel out and pull down. Be careful not to drop panel.



tool provided:
suction cup

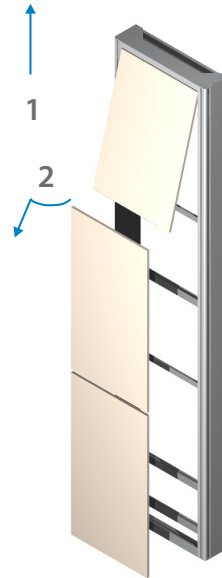


2. Place the chase at its mounting location. Refer to drawings. Mark lines where chase rear brackets are located and where studs intersect. Mount the chase on the wall using hardware compliant to applicable building codes (supplied by others). Use rear brackets to secure chase to wall (brackets to be drilled on site). Ensure chase is secured to wall structure (studs).

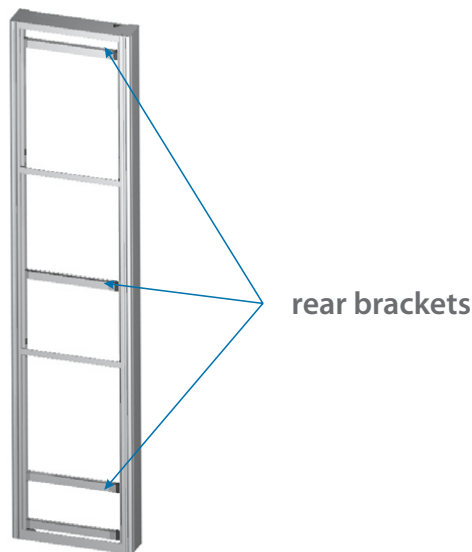

3. Replace laminate panels removed in **step 1**.

Joining the Service Chase with a Single Tier Horizontal Headwall


1. Remove laminate panels from chase to allow access to area where headwall will attach (see chase instructions). Remove fascia, top and bottom covers from headwall to allow access for attaching to chase (see headwall instructions).
2. Attach chase to headwall using hardware supplied as shown:

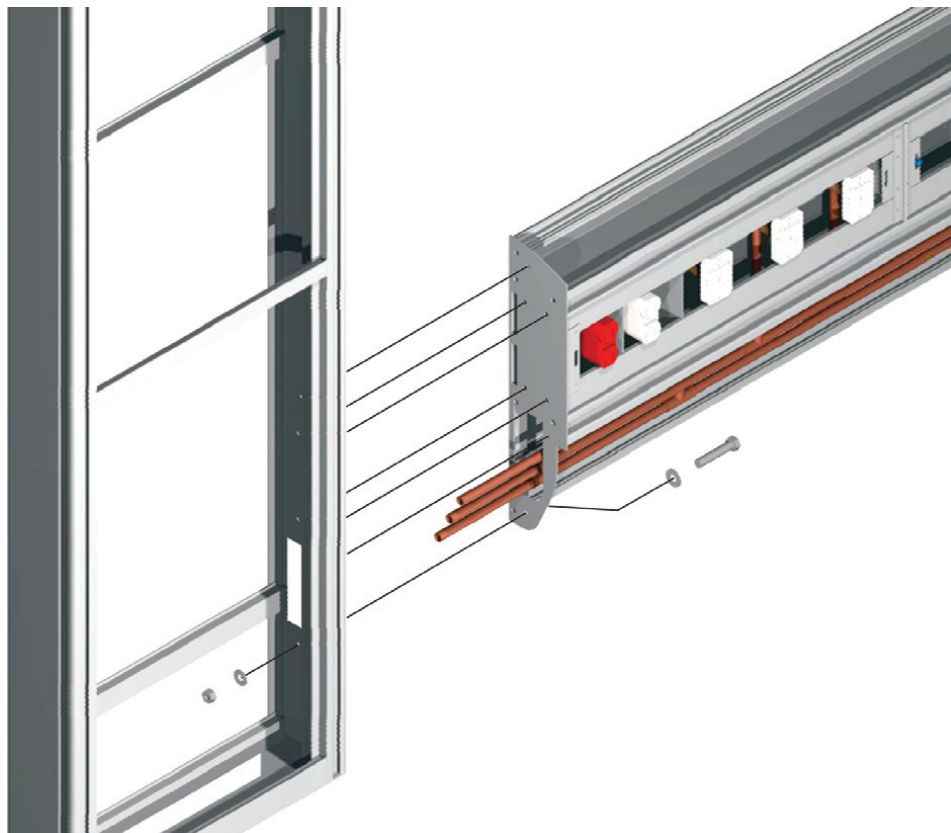

3. Once all electrical and medical gas work is completed, replace fascia, covers and laminate panels.

Joining the Service Chase with a Double Tier Horizontal Headwall

1. Remove laminate panels from chase to allow access to area where headwall will attach (see chase instructions). Remove fascia, top, center and bottom covers from headwall to allow access for attaching to chase (see headwall instructions).
2. Attach chase to headwall using hardware supplied as shown:

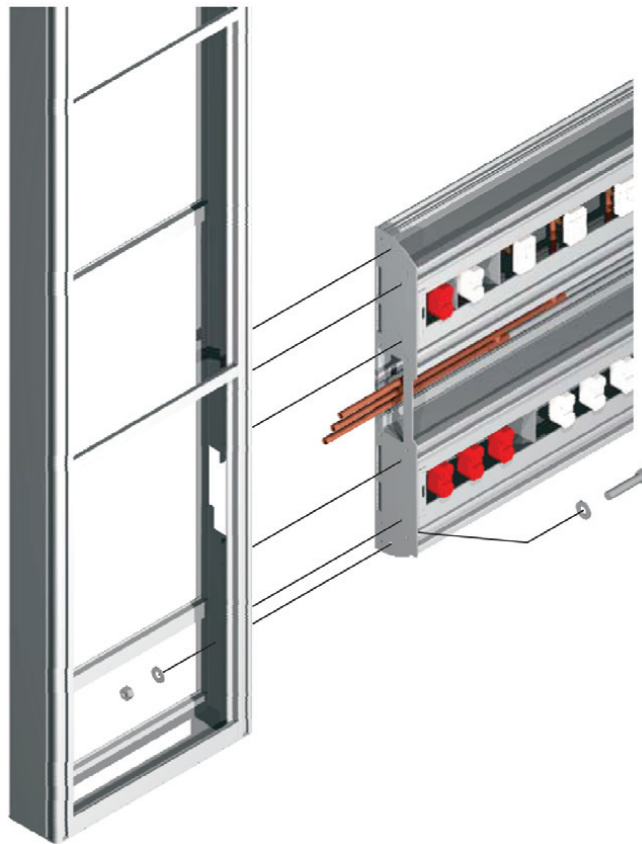

3. Once all electrical and medical gas work is completed, replace fascia, covers and laminate panels.

www.amico.com

Amico Corporation | 85 Fulton Way, Richmond Hill, ON L4B 2N4, Canada
600 Prime Place, Hauppauge, NY 11788, USA
Toll Free Tel: 1.877.462.6426 | Tel: 905.764.0800 | Fax: 905.764.0862
Email: info@amico.com | www.amico.com

