

Direct Mount Aneroid and Cuff Basket Installation Quick Start Guide

Medical Facility Responsibility

Preventive maintenance checks must be performed regularly to maintain the quality and performance of this product. Any parts that may be broken, missing, worn, distorted or contaminated in any way should not be used and all affected parts should be replaced immediately. Should the necessity of any repair be suspected; please contact your local Amico distributor.



Installation Tools

1. 1x Direct-mount Aneroid
2. 4x #8x1" Sheet Metal Screws
3. 4x Plastic Wall Anchors
4. 1x Blood Pressure Cuff Basket
5. 3x #8x1" Sheet Metal Screws
6. 3x #6-8 Plastic Wall Anchors
7. 1x Plastic Clip for 1/4" Wire
8. 1x Blood Pressure Cuff & Bulb, with Coil

Installation Tools



#2 Phillips Head Screwdriver



Drill



1/8" Drill Bit

Installation to Wall

Step 1

Securely attach wall bracket to flat surface at desired height using included screws and wall anchors.

Step 2

Attach clock gauge to wall bracket. Tighten securely using screw driver.

Step 3

Securely attach cuff basket to flat surface using included screws and wall anchors. Basket can be mounted below or beside the aneroid, depending on end-user preference and space availability.

Step 4

Pressure fit "female" end of BP Cuff & Bulb assembly over the "male" nipple just behind the aneroid face.

