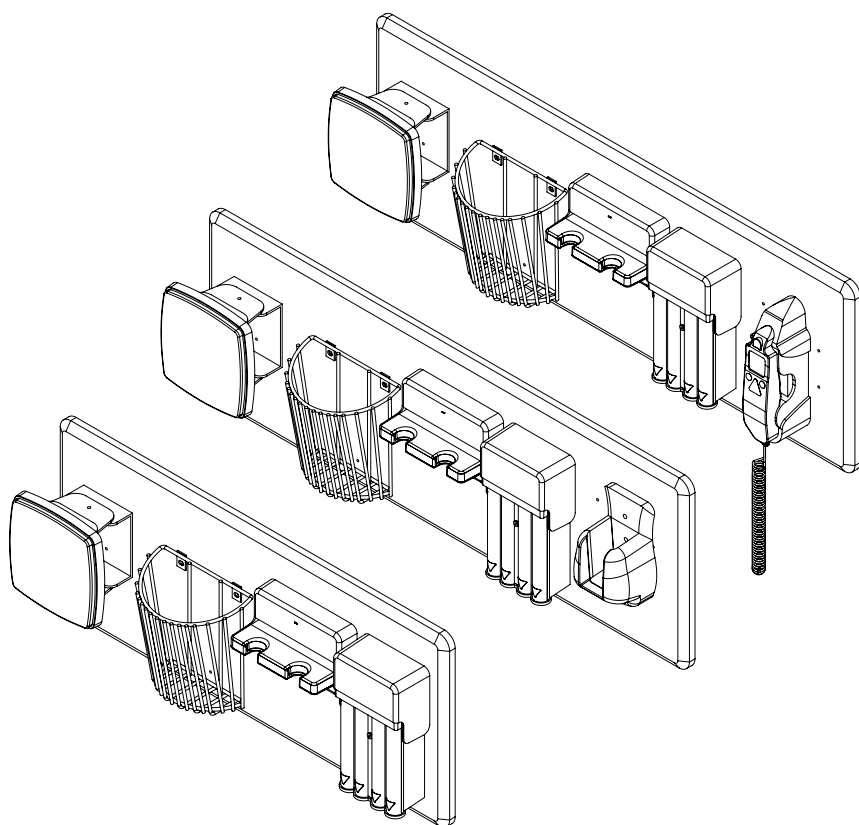
Instructions for Use
Wall Board

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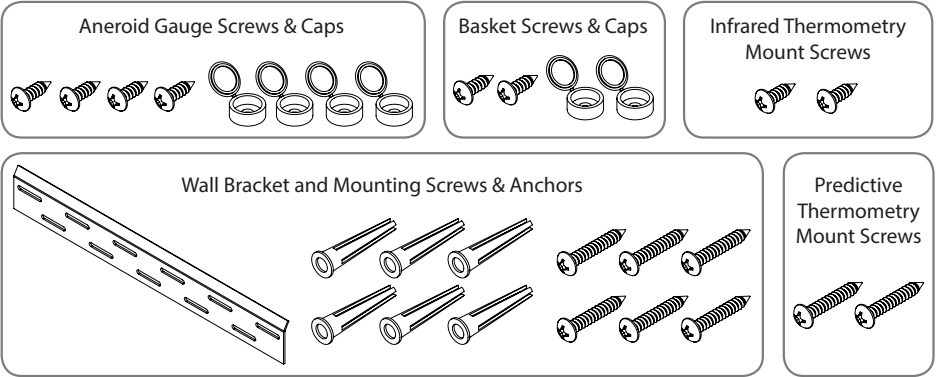
Wall Board Installation Manual

⚠ CAUTION: Amico Diagnostic Incorporated is not responsible for the mounting or integrity of a mounted wall board. It is recommended that the user contacts their Bio Medical Engineering Department or maintenance service to ensure professional installation.

Included Hardware


Wall Bracket Installation

1. Choose desired location for mounting the wall board. A location near the examination table and an electrical outlet is ideal.
2. Locate and mark the stud locations.

For steps 3 through 5, when securing the mounting bracket to a wall with metal studs, replace the drywall screws and anchors with sheet metal screws. When mounting the bracket to a concrete block wall, replace the drywall screws and wall anchors with masonry screws.

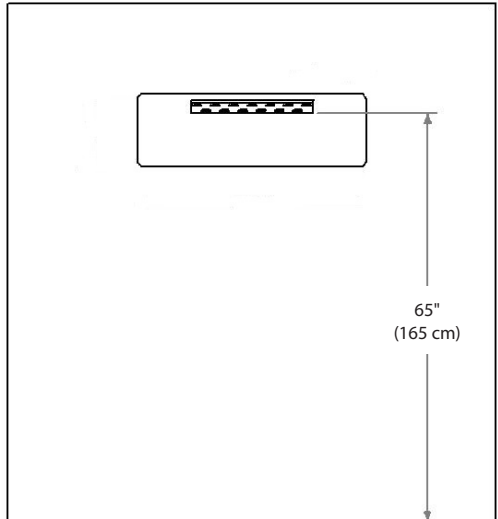3. It is suggested the wall bracket be installed 65" (165 cm) from the floor as shown in diagram. Hold the mounting bracket against the wall and mark the locations for at least four fasteners. It is suggested that one of the markings be aligned with a stud.
4. Remove the mounting bracket. At markings with no stud, drill a 3/16" (0.48 cm) hole for the wall anchors.
5. Insert anchors into holes until flush.
6. Hold the mounting bracket in marked location making sure it is level. Insert screw and tighten until mounting bracket is held tightly.
7. Make sure the mounting bracket is firmly in place.




Installation of Devices


1. The wall board has been pre-drilled for ease of installation. Be sure to align each device with the correct holes.
2. Mount the aneroid gauge to wall board with four screws and four caps. Rotate gauge face to access screw holes. (Figure 1)
3. Insert the bottom rung of the basket into the plastic hook pre-installed on the wall board. Use two screws and two caps to secure the basket to the wall board. (Figure 1)
4. If the wall board contains a thermometer:

Predictive Thermometer - Secure the thermometer mount using the predictive thermometry screws provided. Do not overtighten. (Figure 2)

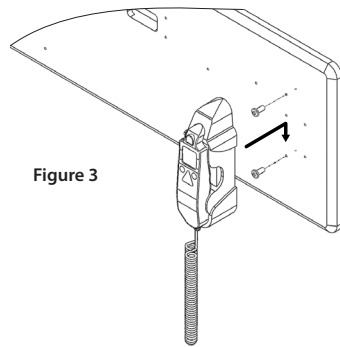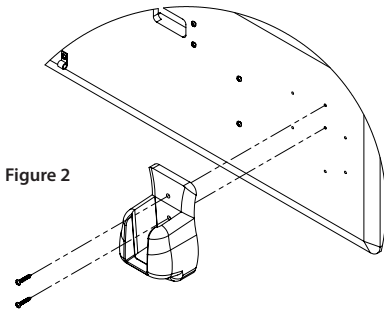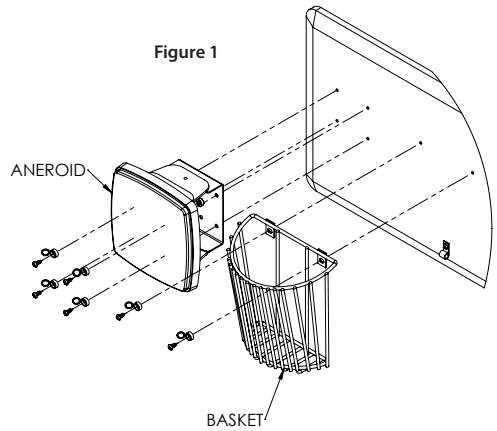Infrared Thermometer - Insert the infrared thermometry screws into the appropriate holes leaving the threads exposed. The infrared thermometer mount can now slide onto the screw heads. (Figure 3)

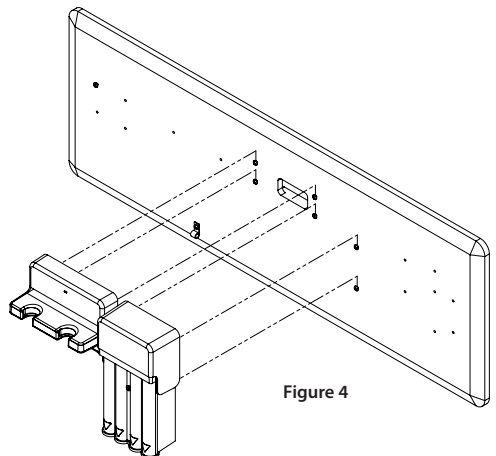5. The power supply, if desired, can be run in the grooves behind the board to the opening. Be sure to leave enough cord in the opening to connect the diagnostic station.
6. The screw holes for the diagnostic station and specula dispenser have been pre-installed on the wall board to the correct height. Align the openings on the back of the devices with the screws and push downwards to secure. (Figure 4)
7. The wall board can now be hung on the bracket. Be sure the bracket is fully sitting within the groove on the back of the wall board.

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