


Introducing the **Sentinel** Vacuum Regulator

THE ONLY REGULATOR
THAT DETECTS
CONTAMINATION!

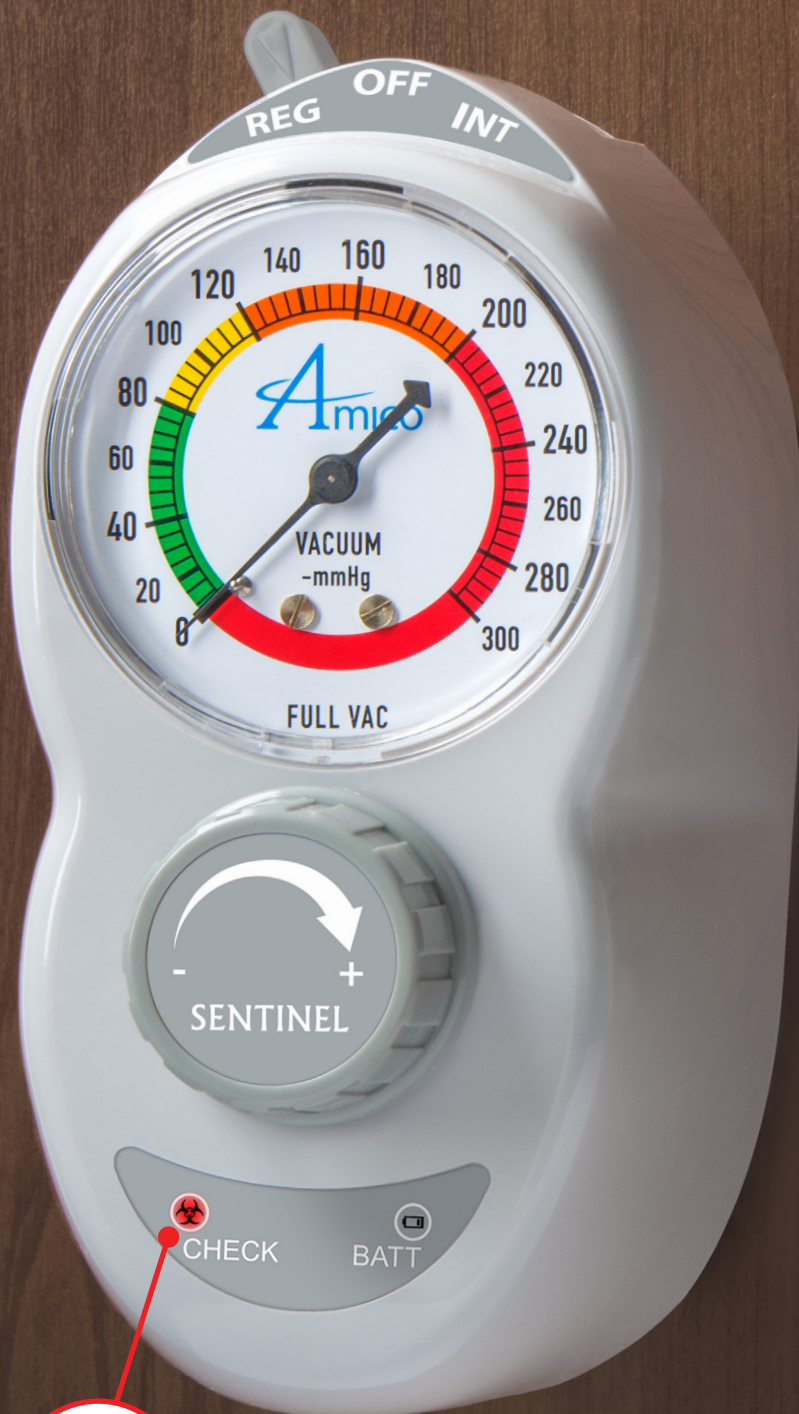



Detects
Contamination



Amico

How Contamination Spreads


- **37% of regulators** in hospitals are **contaminated***
- Contaminants include “growth of well-established nosocomial pathogens” **known to cause Hospital Acquired Infections (HAI)***
- Contaminants **spread from a regulator to the wall-side canister and back to a patient’s stomach within 24 hours***

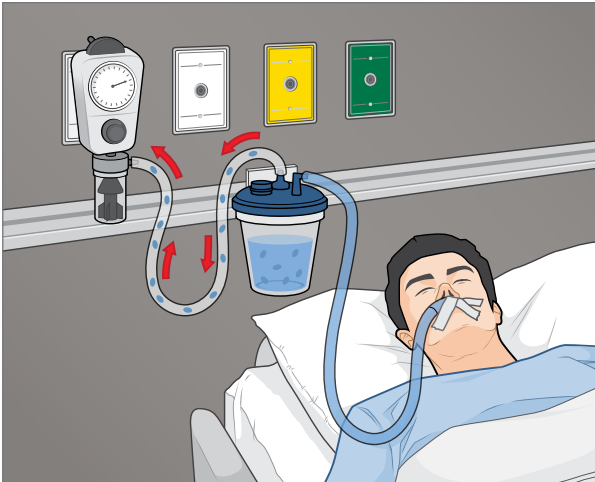

Suctioned fluid flows from patient to canister. Over time, particulates can accumulate in the regulator.



New patient

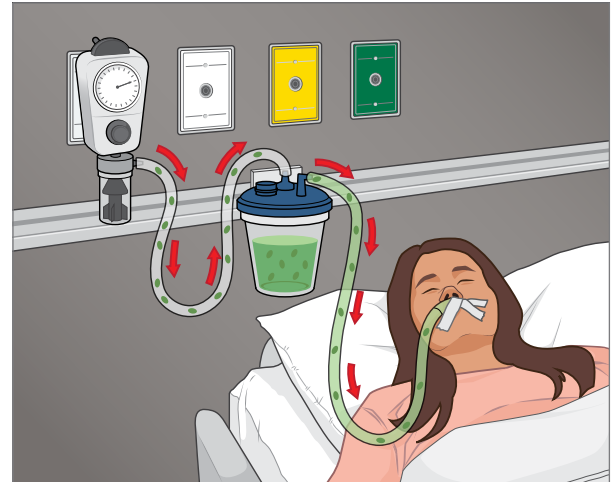

Once contaminated, a new patient can be exposed to bacteria from the contaminated regulator.

* Suction Regulators: A Potential Vector for Hospital Acquired Pathogens by Keith S. Kaye, MD, MPH; Dror Marchaim, MD; Chester Smialowicz, MD; Lauren Bentley, MSBME




the Spread of Contamination with Amico's Sentinel Regulator



Particulates accumulate in the regulator



Contamination occurs

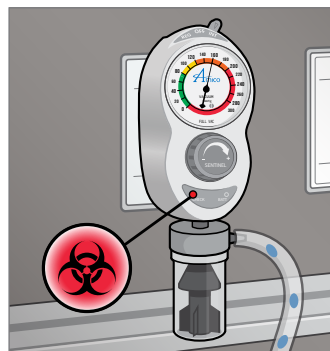


DETECTION

Reverse flush contaminated body



Replace it with a new body

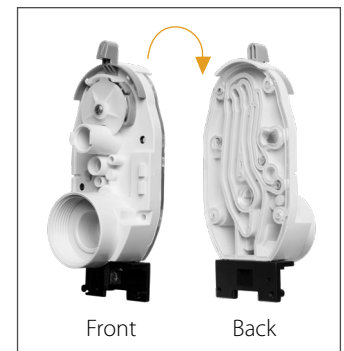

Replaceable body

What To Do When Contaminated

Step 1

Contamination is Detected



Step 2

Remove Contaminated Body



Step 3


Reverse Flush with a Cold Sterilant

OR Discard Contaminated Body



Step 4

Reassemble with only 1 NEW Component

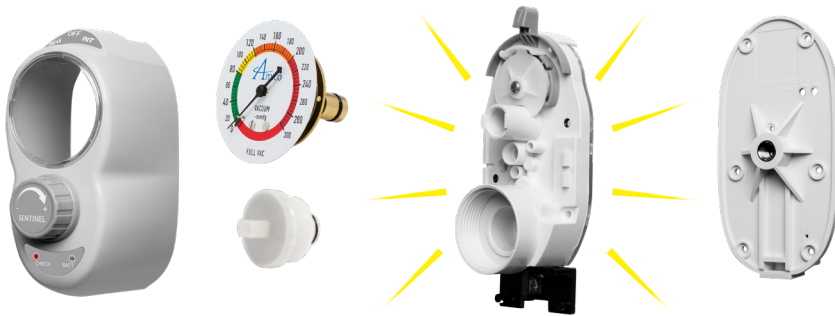

Step 5

Reset the Regulator



No Risk • No Cleaning • Low Cost

Protect your patients
and your pipeline with
the Amico Sentinel!



Analog



Digital

12YEAR
warranty

The Amico Family of Regulators

The Scout - Clear Back Body for Ease of Identifying Contamination



Continuous Models - White Labels

Continuous
3 Mode



SRA-C3U2-F2-C

Continuous
2 Mode



SRA-C2U2-F2-C

Pediatric
2 Mode



SRA-P2U2-F2-C

Neonatal
2 Mode



SRA-N2U2-F2-C

Continuous
High/Surgical



SRA-CHU2-F2-C

Continuous/Intermittent Models - Grey Labels

Adult
Continuous/Intermittent



SRA-CIU2-F2-C

Pediatric
Continuous/Intermittent



SRA-PIU2-F2-C

Neonatal
Continuous/Intermittent



SRA-NIU2-F2-C

The Sentinel - LED Light Indicator to Detect Contamination



Continuous Models - White Labels

Continuous
3 Mode



SSA-C3U2-F2

Continuous
2 Mode



SSA-C2U2-F2

Pediatric
2 Mode



SSA-P2U2-F2

Neonatal
2 Mode



SSA-N2U2-F2

Continuous
High/Surgical



SSA-CHU2-F2

Continuous/Intermittent Models - Grey Labels

Adult
Continuous/Intermittent



SSA-CIU2-F2

Pediatric
Continuous/Intermittent



SSA-PIU2-F2

Neonatal
Continuous/Intermittent



SSA-NIU2-F2